


Quick Reference Guide

Masimo SafetyNet™ App

WARNING: Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.

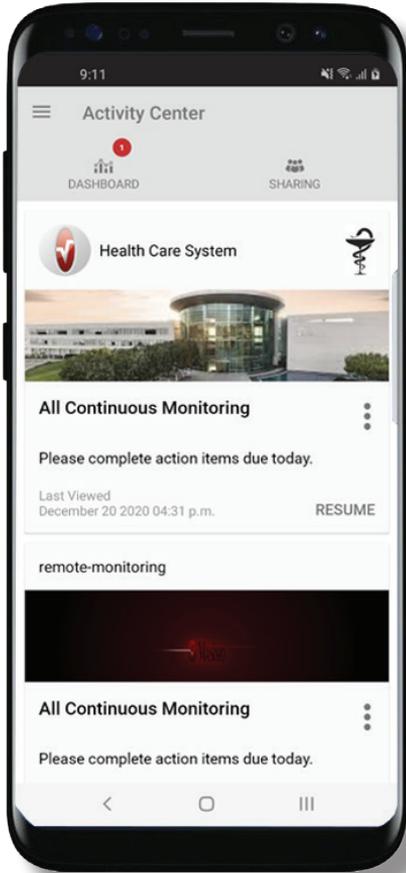

- > Download and install the Masimo SafetyNet App from:

Google Play



The App Store



- > Launch the app and follow the step-by-step instructions to create an account, engage in a shared CareProgram™, and begin collecting and sharing data.

For further assistance or questions visit:

www.masimo.com/masimo-safetynet

©2021 Masimo Corporation. All rights reserved. All other trademarks and registered trademarks are property of their respective owners.




Log In, Sign Up, Location, Notifications

> Sign Up


- Sign up for an account using an email address or phone number.
- If you received a CareProgram from a health provider, you must sign in with the email or phone number used to send your invitation.

> Ensure Location and Notifications are allowed for proper app function.

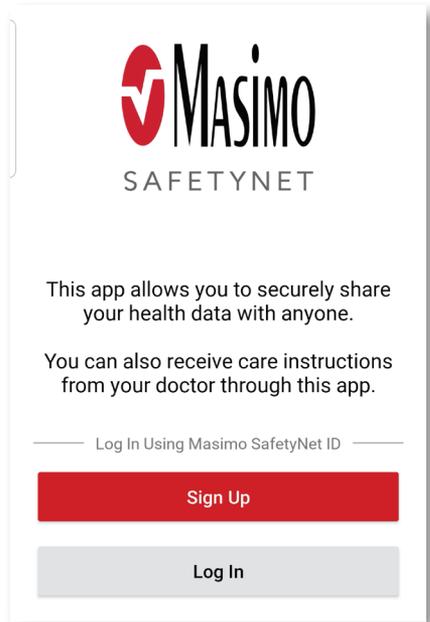


Health Permissions (iOS Only)

iOS

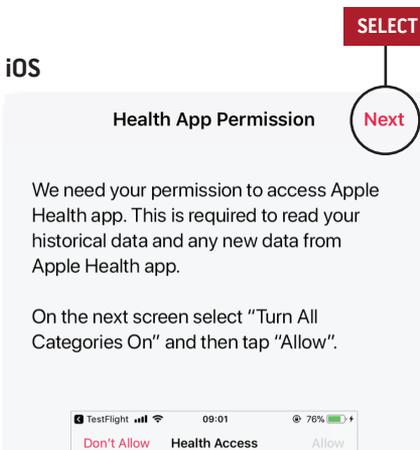

- > **Follow the on-screen instructions:** To allow Masimo SafetyNet to access and gather health data from the device.

Activity Center

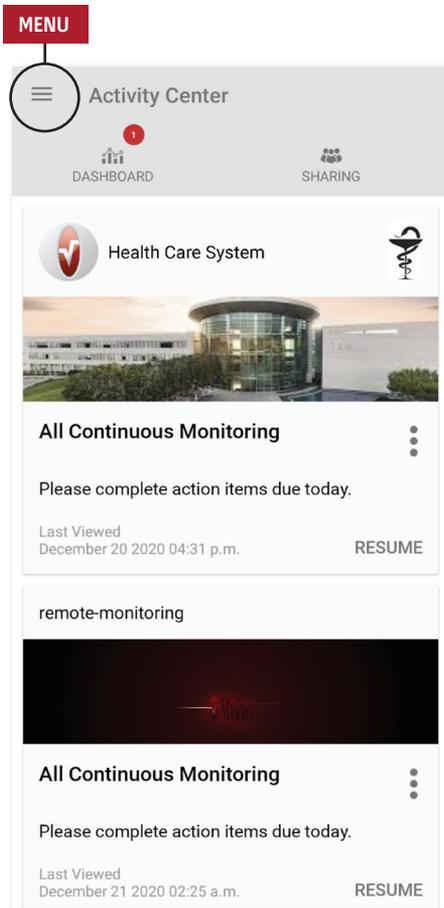

> Menu

- View the App Menu.

> Dashboard

- View the Dashboard screen, which shows the user's health data.
- View health data shared by other users.

> Sharing

- Displays the healthcare providers, caregivers, and family members with whom the user is currently sharing data.
- Share data with another healthcare provider, caregiver, or family member.

> CareProgram

- Allows user to view or delete a CareProgram.

Menu

- The Menu can also be viewed by swiping across the screen from left to right.

> Activity Center

- Activity Center screen.

> Dashboard

- Dashboard screen.

> Profile

- User profile information.
- Modify the profile information.

> My Device

- Add a device for monitoring.

> Live

- View data from the connected device.

> Change Password/Change Security Questions

- Change the password and/or security questions.

> Notification Settings

- Change the app notification and reminder settings.

> Measurement Settings

- Change the units of measurement.

> Settings


- Display user ID.
- Enable or disable Touch ID or Face ID to log into app.
- App information.

> Feedback


- Send an email to Masimo SafetyNet support.

> Log Out

- Log out of the App.

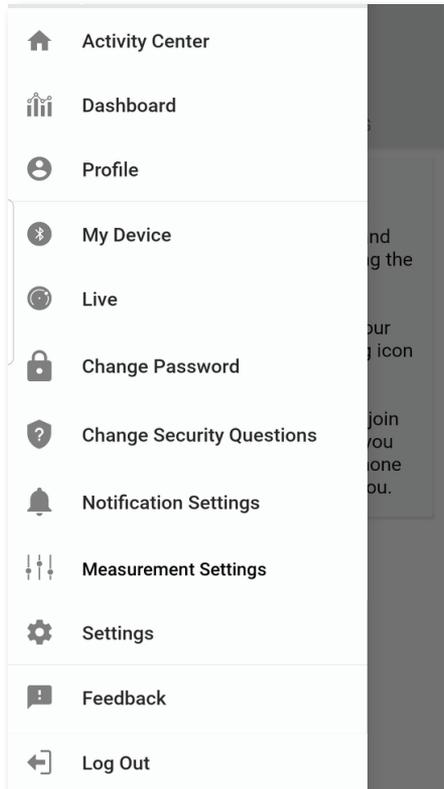

CareProgram

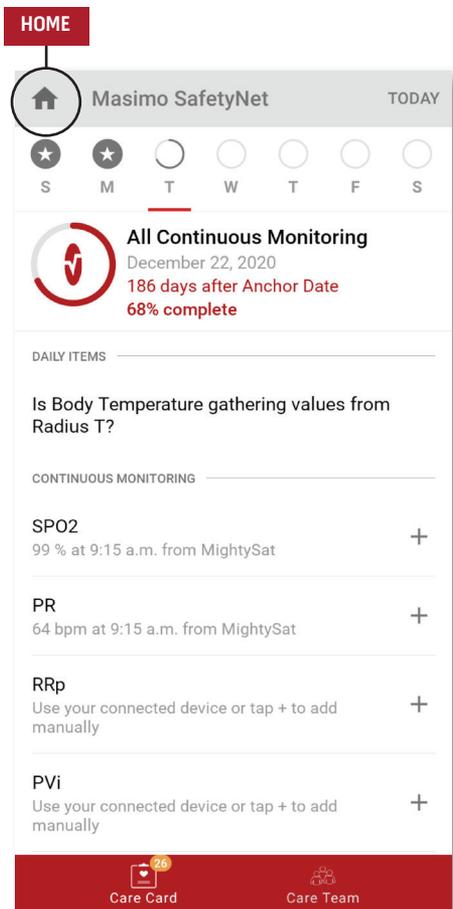

> Care Card

- View Care Card data.

> Home

- Return to the Activity Center.

> Learn

- Located at the bottom of the CareProgram, view additional information provided by a caregiver.


> Care Team

- View list of shared healthcare providers, caregivers, and family members.

Add Device

> Pair a Device


- Select a sensor or device to pair and follow the on-screen instructions.
- Refer to the sensor or device user's manual for additional information.

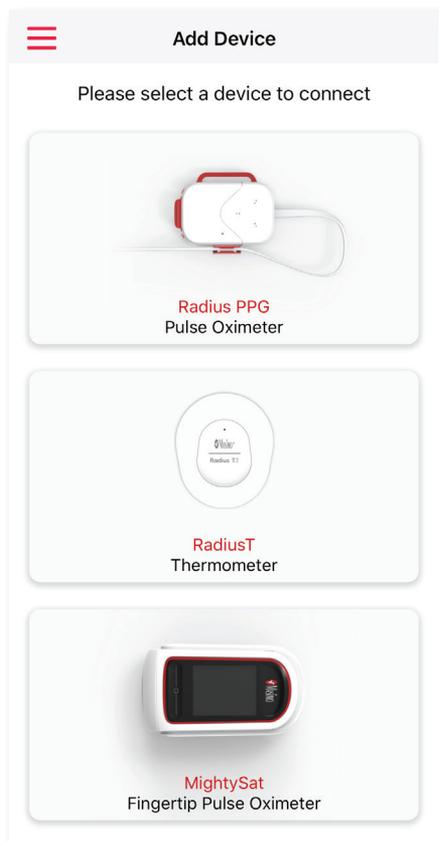

- > If a sensor becomes disconnected, a notification displays. Follow the troubleshooting steps.


Dashboard

- > **Invite**
 - Invite others to share information through the app.
- > **Share**
 - Displays healthcare providers, caregivers, and family members with whom the user is currently sharing data.
- > **Edit**
 - Rearrange data on the Dashboard.
- > **View data for the last 30 days or a weekly summary.**
- > **Select a parameter card to view trends, date ranges, and additional data.**

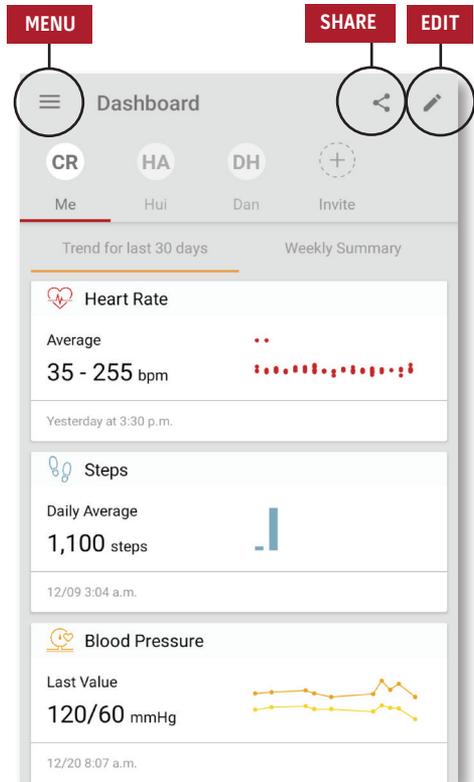


Share

- > **Displays the list of shared healthcare providers, caregivers, and family members.**
- > **Add**
 - Add contact to share with.
- > **Swipe a shared individual to remove or edit privileges.**

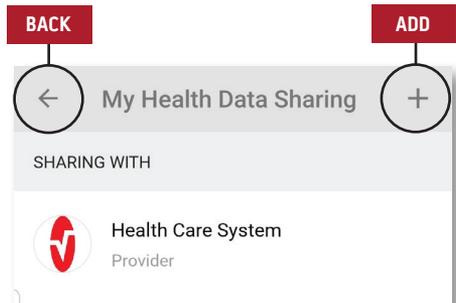

Live View

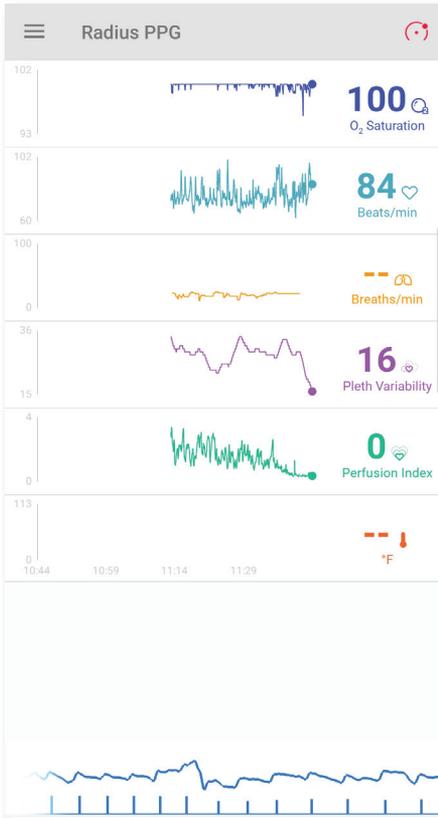

> Live Data

- Displays the live trend view from the connected device.

> History

- Display an hour worth of historical data.

- > Shared users have access to view live data.