



Masimo SafetyNet Alert™

Oxygen Monitoring & Alert System



Quick Start Guide

© 2021 Masimo. All rights reserved. 302386/11129E-1021

SYSTEM COMPONENTS




Masimo Chip

- Reusable chip snaps into the sensor and communicates with the Home Medical Hub.



Masimo SafetyNet Alert App

- Displays oxygen, pulse rate and other data.
- Cloud sends alerts to Home Medical Hub and mobile device.
- Compatible with most mobile devices. (Mobile device not included.)

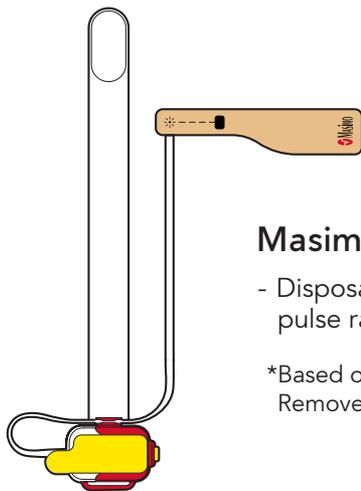

Masimo Sensor


- Disposable sensor measures oxygen level and pulse rate and lasts 7 consecutive nights.*

*Based on 96 hours of sensor battery life. Remove chip between uses.



Home Medical Hub

- Collects data from Masimo Sensor.
- Sends data to smartphone app.

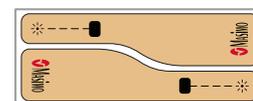

Replacement Tapes


- Additional tapes for replacement as needed.

SET UP APP

1. Download app.



2. Connect Home Medical Hub to power.

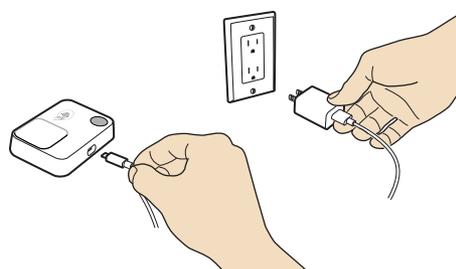

3. Look for power indicator light.




4. Follow step-by-step instructions on app to set up system.




SET UP SENSOR

2


1. Remove sensor from pouch.

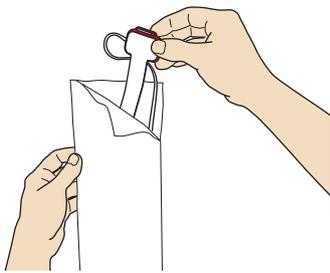


2. Peel off yellow label to activate battery.

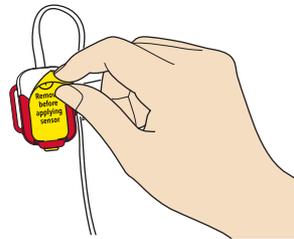

3. Place sensor on wrist as shown and carefully thread the tip of the strap through the open red loop.

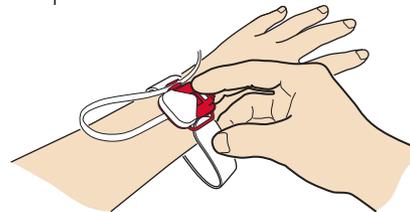


4. Wrap the loose end of the strap around your wrist and press to secure.

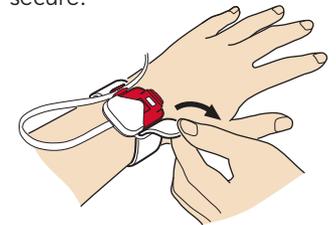


APPLY SENSOR

3


1. Remove part of the plastic film and place star (✱) on top of fingernail of ring finger.

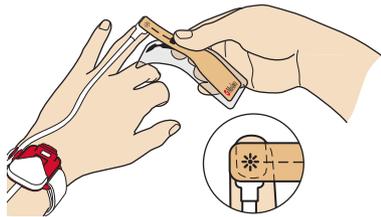


2. Wrap tape around finger so black square (■) is on finger pad directly across from star (✱).

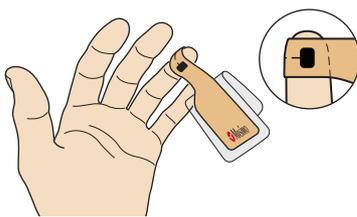


3. Squeeze top and bottom of finger to secure tape.

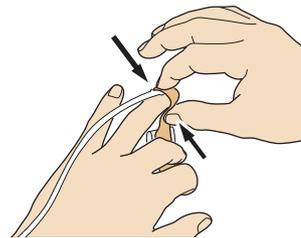

4. Remove plastic film and continue to wrap tape around finger.

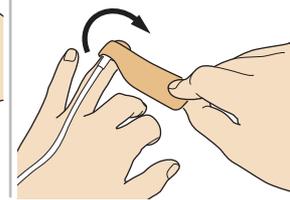


5. Adjust cable to comfortable length.

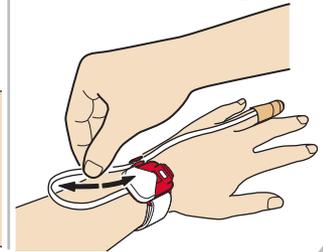


PAIR AND INSERT CHIP

4




1. Hold the chip near the Home Medical Hub and listen for double beep.

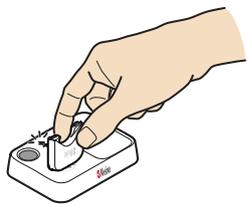

2. Insert the chip into open compartment on the sensor.

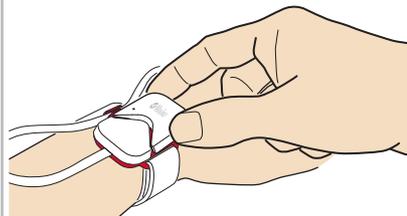

3. Press down on the chip to secure.

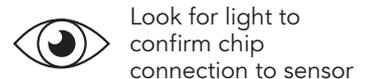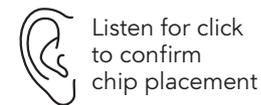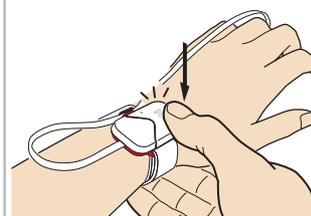

CONFIRM SUCCESSFUL PAIRING

5


1. Once the chip is inserted, the Home Medical Hub Bluetooth Status LED will turn off and the Status LED will turn green. The chip light indicator will turn solid blue.




Visit <http://www.mymasimo.co.uk/safetynet-alert> to see full list of supported devices, purchase additional sensors and view other resources, including videos and instructions.

CHANGING THE REPLACEMENT TAPE


1. Remove used tape from sensor

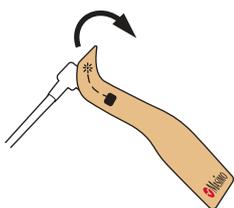

2. Attach new tape in same direction as used tape

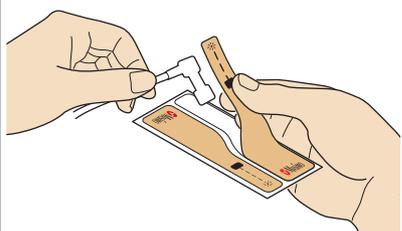

3. Attach sensor to finger

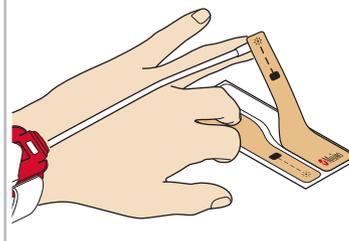

4. Peel the remaining tape off backing

